

Save money

by switching to

Pylobactell

¹³C-UREA BREATH TEST (UBT)

for your *H. pylori* testing requirements



The lowest cost UBT available*



Express analysis & results
>95% of tests reported on day of receipt



The only UBT licensed for patient home use

Releasing your valuable clinical time



For the reliable and fast *in-vivo* detection of the presence of *Helicobacter pylori*

Approved in all EU countries – licensed as a prescription only medicine

www.breathlab.com

Pylobactell

¹³C-UREA BREATH TEST (UBT)

...breathtakingly precise

Order directly from:

**Torbet Laboratories Ltd, Unit 1 Chestnut Drive,
Wymondham, Norfolk, NR18 9SB.**

Tel: 01953 607856 Fax: 01953 713649

E-mail: pylobactell@breathlab.com

www.breathlab.com

Abridged Prescribing Information – Pylobactell® (13C-Urea):

Presentation: Soluble tablet containing 100mg ¹³C-Urea. **Uses:** In vivo diagnosis of gastroduodenal *Helicobacter pylori* infection. **Dosage and Administration: Adults:** The tablet is to be dissolved in water and taken 10 minutes after the start of the breath test procedure. The patient should fast for at least 4 hours before the test so that the test is taken on an empty stomach. If the patient has eaten a heavy meal, then it will be necessary to fast for 6 hours prior to the test. **Paediatric patients:** Pylobactell is not recommended for use in children and adolescents below the age of 18 years due to insufficient data on efficacy. **Contraindications:** Hypersensitivity to the active substance or to any of the excipients. The test must not be used in patients with documented or suspected gastric infection that might interfere with the urea breath test. **Special Warnings and Precautions for Use:** A positive urea breath test alone does not clinically confirm that eradication therapy is indicated. Alternative diagnosis with invasive endoscopic methods might be indicated in order to examine the presence of any other complicating conditions, e.g. gastric ulcer, autoimmune gastritis and malignancies. In individual cases of atrophic gastritis, the breath test result may have a false positive outcome and other tests may be required to confirm the presence of *H. pylori*. If a repeat test is required, it should not be carried out until the following day. For patients who do not tolerate the recommended test meal, an alternative test meal should be given. Care should be taken in patients where fasting may have medical implications. There are insufficient data on the diagnostic reliability of the Pylobactell test to recommend its use in patients with partial gastrectomy and in patients younger than 18 years. **Side Effects:** None known. **Instructions for Use and Handling:** Fasting (see dosage and administration). The patient should be in a seated position. T=0: Drink test meal, T=5 mins: Collect Pre-urea samples, T=10 mins: Drink dissolved PYLOBACTELL® tablet, T=40 mins: Collect Post-urea samples. Transfer spare bar code labels to patient's notes and analysis request form. Seal box with the security label and send to a qualified laboratory for analysis. Legal Category: POM. **NHS Price** £20.75. **Marketing Authorisation Number:** EU/1/98/064/001. **Marketing Authorisation Holder:** Torbet Laboratories Ltd. **Date of Preparation:** March 2013. **Full prescribing information is available from Torbet Laboratories Limited. Adverse events should be reported. Reporting forms and information can be found at www.mhra.gov.uk/yellowcard. Adverse events should also be reported to Torbet Laboratories Limited.**

***Based on January 2017 price lists**